
Guark C12x-T12x

Stress Testing ECGs

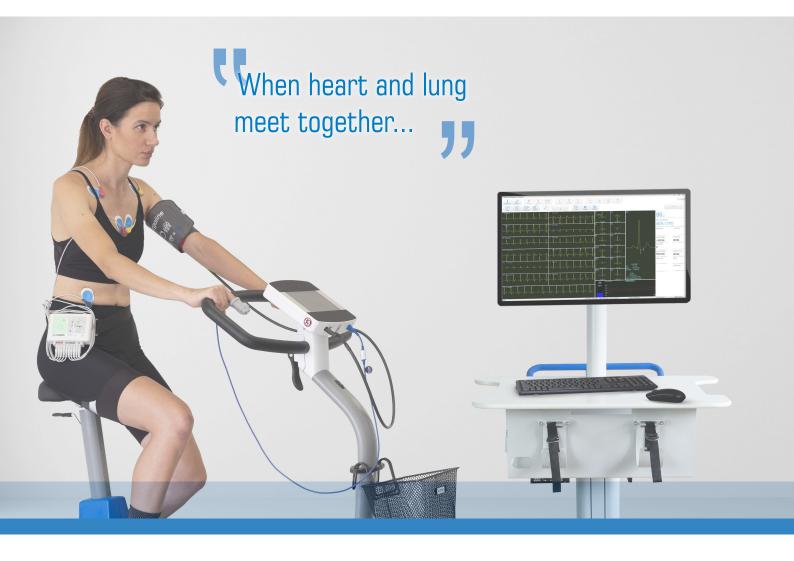

Diagnostic Quality 12-lead Stress Testing ECGs available in telemetry or cable configuration



Electrocardiographic (ECG) recording system for continuous monitoring of heart rhythm and evaluation of ischemic ECG changes during exercise and recovery⁽¹⁾

- Resting and Exercise ECG interpretation
- | High resolution ECG processing
- | Full Disclosure and Scroll back during the test
- | ST segment, ST slopes, Trends
- Retrospective ECG and arrhythmia analysis even during test
- Standard and user defined Exercise Protocols
- Integration with COSMED Gas exchange equipment
- Patient Database & Network Compatibility

COSMED Stress ECGs are the ideal diagnostic quality PC-based ECG devices for patient care from prevention to rehabilitation. Ultimate quality ECG signal allows an efficient diagnosis of coronary artery disease, the evaluation of patients after artery bypass surgery or angioplasty, and allows to document response to therapies and prevent from myocardial infarction.

Combined with any COSMED gas exchange analysis equipment, COSMED Stress ECGs become the complementary tools that help clinician to quantify patient's functional capacity and evaluate cardiac response in athletes.


COSMED PC-based stress testing systems offer the ultimate in stress testing flexibility, meeting the most advanced requirements for clinical exercise testing.

Two versions are currently available:


- Quark C12x 12-lead Patient Cable Stress Test ECG, with USB communication interface.
- Quark T12x 12-lead Wireless Stress Test ECG (Telemetry). State of the art Digital Radio Frequency technology, immune to environmental interference. Optimized for working in noisy EM Environments (Bluetooth, Wi-Fi Networks, Cellphones etc.). It includes an automatic Channel Shift Feature for selecting the optimal Transmission Channel.

Design

- 12 leads ECG 12-lead simultaneous recording, display and printout. Detachable 10 lead wires conform to AAMI (AHA) or IEC color codes. Every single lead can be replaced independently
- High resolution of signal processing (up to 500 samples/sec) guarantees exceptionally clear tracing for visual view and on screen measurements
- On-line filtering Advanced digital filters and automated baseline correction for realtime and stored ECGs to create exemplary ECG traces
- Integration with COSMED gas exchange equipment stress ECG and ergospirometry parameters can be thus recorded synchronously, identifying cardiac, pulmonary and vascular function in just a single measurement

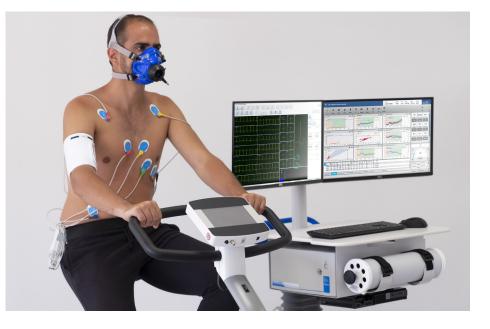



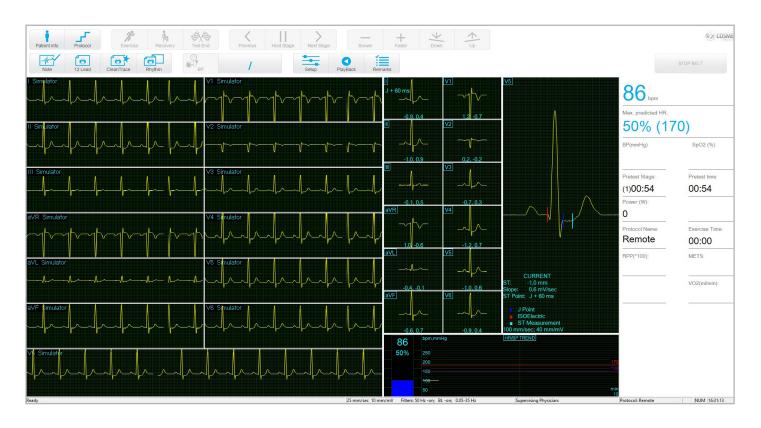

Software Features

Powerful software allows full control over ECG acquisition, real time display, report configuration and printing.

High resolution ECG processing produces an exceptionally clear on-screen display and allows detailed, reliable analysis of ST segments and subtle arrhythmia changes.

Main software features are:

- Full disclosure of entire test enabling post processing on saved data
- Scroll-back during test to see episodes that might have been overseen
- Automatic Arrhythmia detection, print and capture with the possibility to disable unwanted detection types

- Pacemaker detection with mark spikes on screen and in report
- Real time and retrospective J Point and Isoelectric identification
- Predicted oxygen uptake and METS estimation
- Real time super imposition QRST complex
- Remote viewing anywhere on your local hospital network room
- User defined, automatic print during test (standard paper or thermal paper print options)
- Complete reporting functionality allowing the configuration of textual summary and ECG data


- Quality controls including lead quality check and audible signals for lead disconnection and max HR
- User initiated and automatic capture of events
- Pre-programmed user definable testing protocols, including pharmacologic test profiles and many different treadmills and bicycle ergometers
- Export formats: JPEG, XML, GDT, PDF
- Integration with OMNIA Suite including database
- Multiple SpO2 sensors integration



Nonin[®] Wrist Ox2 3150 pulse oximeter for SpO2 integration



Suntech Tango[®] blood pressure monitor allows to integrate several parameters, including SpO2

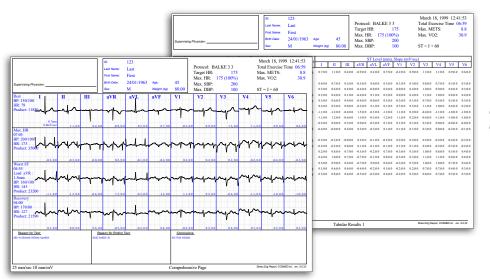

Multiple printout report formats (single lead, all leads, full disclosure, sum up, trends, QRS, etc.)



Headquarters

COSMED Srl Rome +39 06 931-5492 info@cosmed.com

GERMANY

SWITZERLAND

Fehraltorf

COSMED Deutschland GmbH Werneck +49 (0)9735 81390 00 DE@cosmed.com

COSMED Switzerland GmbH

+41 (0)43 50 869 83 CH@cosmed.com

FRANCE

USA

COSMED France SASU Brignais +33 (0)4 478628053 FR@cosmed.com

COSMED USA, Inc. Concord, Chicago

USA@cosmed.com

+1 800 4263763 Toll Free

THE NETHERLANDS

BNL@cosmed.com

AUSTRALIA

Artarmon

+61 449 971 170

ANZ@cosmed.com

COSMED Benelux BV Nieuwegein +31 (0) 88 10 50 500

DENMARK

COSMED Nordic ApS Odense +45 6595 9100 DK@cosmed.com

HONG KONG

COSMED Asia-Pacific Pty Ltd Kowloon

COSMED HK Ltd

+852 3708 3126 HK@cosmed.com

To know more



COSMED Srl

Albano Laziale - Rome 00041 Italy +39 (06) 931-5492 Phone +39 (06) 931-4580 Fax

cosmed.com

Distributed by

Via dei Piani di Monte Savello 37